


**Breathable Train®
Active Knit**
gentle on the skin,
excellent wearing comfort

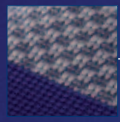


**Three-dimensional
Train® Active Knit**
for a perfect fit



**Less compression
at edges** prevents
constriction



Stretches easily
so support is easy to
put on and take off

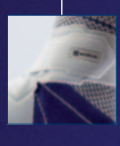

**Individually adjustable
strap system**
dynamic stabilization

What's stopping you from a life in motion?

- You can use MalleoTrain® S preventatively to minimize the risk of injury.
- You don't have to give up sports just because you have a mild sprain or a torn ligament. MalleoTrain® gives your ankle reliable support and stability.

Performance and Prevention

Are your ligaments unstable?

- When they feel unstable, protect and support your ligaments with the MalleoTrain® S active support.

Stability and security when exercising

- The strap is wound around in a stabilizing figure-8 pattern, securing your ankle.
- When wrapping your ankle you can individually adjust the amount of stretch, determining the active support's effect.

Always ready for action...

MalleoTrain® S
can be put on in **15 seconds!**

Comfortable to wear ...

- The air-permeable, breathable knit is gentle on the skin and feels good to wear.
- The active support fits comfortably in all comfort shoes and athletic footwear.





The training partner for your ankle.

MalleoTrain® S

SUPPORTS



The new MalleoTrain® S – goes on quickly and easily!



1 Put the active support on



2 Wrap the strap from the inside of your foot under your foot.



3 Bring the strap diagonally across the top of your foot.



4 Then wrap it around your ankle above your heel ...



5 ... and then again diagonally toward the front over the top of your foot.



6 Wrap the thin part of the strap under your foot to the outside and then diagonally across the top of your foot.



7 Stick the end of the strap to the strap itself.



SUPPORTS AND ORTHOSES
MEDICAL COMPRESSION STOCKINGS
INSOLES AND SHOES
PROSTHETIC COMPONENTS AND MATERIALS
MEASUREMENT TECHNOLOGY

United Kingdom
Bauerfeind UK
Phyllis House
229 Bristol Road
GB-Birmingham B5 7UB
Phone +44 (0) 121 446 53 53
Fax +44 (0) 121 446 54 54
E-mail info@bauerfeind.co.uk

USA
Bauerfeind USA, Inc.
55 Chastain Road
USA-Kennesaw, GA 30144
Phone 1 800 423 34 05
Fax (770) 429 84 77
E-mail info@bauerfeindusa.com